




<b>Stool Testing Universal Laboratory (STUL) Kit (Oracle #1570140)</b>			
<p>Send all (3) kit vials to the laboratory in the kit's white bag or a biohazard bag. Vials must include <b>two patient identifiers</b>. Vials will be saved for add-on testing (stability permitting).</p> <p>*When possible, aliquot excess stool into a refrigerated sterile container, an EcoFix vial, and a C&amp;S vial.</p>			
	EcoFix	C&S/Cary-Blair	Sterile Container*
Cryptosporidium & Giardia Antigens by EIA ( <b>OVAPSC</b> )	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<b>Preferred*</b> <small>*An EcoFix vial (left) is also needed for reflex CRYSCO to confirm Cryptosporidium-positive EIAs.</small>	<input checked="" type="checkbox"/> <b>Acceptable</b>
Ova & Parasite Examination ( <b>OVAP</b> )	<input checked="" type="checkbox"/> <b>Preferred</b>	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Acceptable</b>
<b>OVAP can not identify Cryptosporidium, Cyclospora, Cystoisospora, and Microsporidia; CRYSCO and MICSP0 must be ordered.</b>			
Cryptosporidium, Cyclospora, Cystoisospora Examination ( <b>CRYSCO</b> )	<input checked="" type="checkbox"/> <b>Preferred</b>	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Acceptable</b>
Microsporidia Examination ( <b>MICSP0</b> )	<input checked="" type="checkbox"/> <b>Preferred</b>	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Acceptable</b>
Enteric Bacterial Panel by PCR ( <b>STLPCR</b> ) and Enteric Viral Panel by PCR ( <b>EVPPCR</b> )	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Preferred</b>	<input checked="" type="checkbox"/> <b>Acceptable</b>
Clostridium difficile (C. Diff) Toxin by PCR ( <b>CDPCR</b> )	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Preferred</b>
Stool Gastrointestinal Panel by PCR ( <b>STGIPI</b> )	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Preferred</b>	<input checked="" type="checkbox"/> <b>Unacceptable</b>
Helicobacter pylori Antigen by EIA, Stool ( <b>HPYLAG</b> )	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Preferred</b>
Fecal Lactoferrin ( <b>FECWBC/STLWBC</b> )	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Unacceptable</b>	<input checked="" type="checkbox"/> <b>Preferred</b>

**Other/Miscellaneous Stool Testing**

Fecal Occult Blood Test (**IFOBT**) (Oracle #1045438)

Collection & Transport Item:  
**OC-Auto® FIT Personal Use Kit**



Pinworm Preparation (**TAPE**) (Oracle #1621278)

Collection & Transport Item:  
**Pinworm Paddle**

