





Collection Kit	Sample Type	Gonorrhea & Chlamydia	<i>Mycoplasma genitalium</i>	<i>Trichomonas vaginalis</i>	Candida & Trichomonas	Bacterial Vaginosis
		GCCT	MYGAMP	TRVAMP	CVTV	BVAMP
 <p><b>Aptima® Multitest Swab</b> Oracle #1065401</p>	Clinician-Collected Vaginal Swab	✓✓✓	✓✓✓	✓✓✓	✓✓✓	✓✓✓
	Patient-Collected Vaginal Swab	✓✓✓	✓✓✓		✓✓✓	✓✓✓
	Patient-Collected Penile Meatal Swab		✓✓			
	Clinician-Collected Throat Swab	✓✓✓				
	Clinician-Collected Rectal Swab	✓✓✓				
 <p><b>Aptima® Unisex Swab</b> Oracle #1075104</p>	Clinician-Collected Endocervical Swab	✓✓	✓	✓✓		
	Clinician-Collected Male Urethral Swab	✓✓	✓✓	*		
 <p><b>Sterile Container</b> Oracle #1012787</p>	Female Urine*	✓	✓	*		
	Male Urine*	✓✓✓	✓✓✓	*		
 <p><b>ThinPrep® &amp; Collection Device</b> Oracle #1060033 &amp; 4</p>	ThinPrep® (PreservCyt®)** <small>Only orderable as an add-on test by calling Lab Client Services</small>	✓		✓		
✓✓✓	✓✓	✓		*		
FDA-approved & preferred	FDA-approved & acceptable	FDA-approved, but not recommended due to inferior performance			Lab-Developed Test (not FDA-approved)	

\***LAB ONLY:** Within 24 hours of collection, aliquot the specimen from the sterile container into a **yellow Aptima® Urine Tube** (Oracle #10444480). Do not overfill or underfill.

\*\***LAB ONLY:** Prior to cytology processing, aliquot the specimen from ThinPrep® container into a **green Aptima® Specimen Transfer Tube** (Oracle #1052559).

<b>Additional collection guidelines to avoid specimen rejection:</b>	Match the <b>sample type</b> to the correct <b>collection kit</b> .	<b>Pink swab = orange tube</b> <b>Blue swab = white tube</b>	Discard / <b>do not send the white cleaning swabs</b> to the laboratory.	A maximum of <b>two tests per collection kit</b> are allowed. <ul style="list-style-type: none"> <li>(1) Beaker label per kit</li> <li>(1) swab per kit</li> </ul>
--	---	---	--	--