





Collection Kit	Sample Type	Gonorrhea & Chlamydia	<i>Mycoplasma genitalium</i>	<i>Trichomonas vaginalis</i>	Candida & Trichomonas	Bacterial Vaginosis
		GCCT	MYGAMP	TRVAMP	CVTV	BVAMP
 Aptima® Multitest Swab Oracle #1065401	Clinician-Collected Vaginal Swab	✓✓✓	✓✓✓	✓✓✓	✓✓✓	✓✓✓
	Patient-Collected Vaginal Swab	✓✓✓	✓✓✓	✓✓✓	✓✓✓	✓✓✓
	Patient-Collected Penile Meatal Swab		✓✓			
	Clinician-Collected Throat Swab	✓✓✓				
	Clinician-Collected Rectal Swab	✓✓✓				
 Aptima® Unisex Swab Oracle #1075104	Clinician-Collected Endocervical Swab	✓✓	✓	✓✓		
	Clinician-Collected Male Urethral Swab	✓✓	✓✓			
 Sterile Container Oracle #1012787	Female Urine*	✓	✓	✓✓		
	Male Urine*	✓✓✓	✓✓✓	✓✓✓		
 ThinPrep® & Collection Device Oracle #1060033 & 4	ThinPrep® (PreservCyt®)** <small>Only orderable as an add-on test by calling Lab Client Services</small>	✓		✓		
✓✓✓		✓✓			✓	
FDA-approved & preferred		FDA-approved & acceptable			FDA-approved, but not recommended due to inferior performance	

***LAB ONLY:** Within 24 hours of collection, aliquot the specimen from the sterile container into a **yellow Aptima® Urine Tube** (Oracle #10444480). Do not overfill or underfill.

****LAB ONLY:** Prior to cytology processing, aliquot the specimen from ThinPrep® container into a **green Aptima® Specimen Transfer Tube** (Oracle #1052559).

Additional collection guidelines to avoid specimen rejection:	Match the sample type to the correct collection kit .	Pink swab = orange tube Blue swab = white tube	Discard/do not send the white cleaning swabs to the laboratory.	A maximum of two tests per collection kit are allowed. <ul style="list-style-type: none"> (1) Beaker label per kit (1) swab per kit
--	---	---	--	--