





Collection Kit	Sample Type	Gonorrhea & Chlamydia	<i>Mycoplasma genitalium</i>	<i>Trichomonas vaginalis</i>	Candida & Trichomonas	Bacterial Vaginosis
		GCCT	MYGAMP	TRVAMP	CVTV	BVAMP
 <b>Aptima® Multitest Swab</b>	Vaginal Swab, Patient or Clinician-Collected <i>collected in a healthcare setting</i>	✓✓✓	✓✓✓	✓✓✓	✓✓✓	✓✓✓
	Penile Meatal Swab, Patient-Collected <i>collected in a healthcare setting</i>		✓✓			
	Rectal Swab, Clinician-Collected	✓✓✓				
	Rectal Swab, Patient-Collected <i>collected in a healthcare setting</i>	*				
	Throat Swab, Clinician-Collected	✓✓✓				
 <b>Aptima® Unisex Swab</b>	Endocervical Swab, Clinician-Collected	✓✓	✓	✓✓		
	Male Urethral Swab, Clinician-Collected	✓✓	✓✓			
 <b>Sterile Container</b>	Female Urine*	✓	✓	✓✓		
	Male Urine*	✓✓✓	✓✓✓	✓✓✓		
 <b>ThinPrep® &amp; Collection Device</b>	ThinPrep® (PreservCyt®)** <small>Only orderable as an add-on test by calling Lab Client Services</small>	✓		✓		
✓✓✓	✓✓	✓			*	
FDA-approved & preferred	FDA-approved & acceptable	FDA-approved, but not recommended due to inferior performance			Lab-Developed Test (LDT), not FDA-approved	

\***LAB ONLY:** Within 24 hours of collection, aliquot the specimen from the sterile container into a **yellow Aptima® Urine Tube**. Do not overfill or underfill.

\*\***LAB ONLY:** Prior to cytology processing, aliquot the specimen from ThinPrep® container into a **green Aptima® Specimen Transfer Tube**.

<b>Additional collection guidelines to avoid specimen rejection:</b>	Match the <b>sample type</b> to the correct <b>collection kit</b> .	<b>Pink swab = orange tube</b> <b>Blue swab = white tube</b>	Discard/do not send the <b>white cleaning swabs</b> to the laboratory.	A maximum of <b>two tests per collection kit</b> are allowed. <ul style="list-style-type: none"> <li>• (1) Beaker label per kit</li> <li>• (1) swab per kit</li> </ul>