





Collection Kit	Sample Type	Gonorrhea & Chlamydia GCCT	Mycoplasma genitalium MYGAMP	Trichomonas vaginalis TRVAMP	Candida & Trichomonas CVTV	Bacterial Vaginosis BVAMP
 Aptima® Multitest Swab	Vaginal Swab, Patient or Clinician-Collected <i>collected in a healthcare setting</i>	✓✓✓	✓✓✓	✓✓✓	✓✓✓	✓✓✓
	Penile Meatal Swab, Patient-Collected <i>collected in a healthcare setting</i>		✓✓			
	Rectal Swab, Clinician-Collected	✓✓✓				
	Rectal Swab, Patient-Collected <i>collected in a healthcare setting</i>	*				
	Throat Swab, Clinician-Collected	✓✓✓				
 Aptima® Unisex Swab	Endocervical Swab, Clinician-Collected	✓✓	✓	✓✓		
	Male Urethral Swab, Clinician-Collected	✓✓	✓✓			
 Sterile Container	Female Urine*	✓	✓	✓✓		
	Male Urine*	✓✓✓	✓✓✓	✓✓✓		
 ThinPrep® & Collection Device	ThinPrep® (PreservCyt®)** Only orderable as an add-on test by calling Lab Client Services	✓		✓		
✓✓✓	✓✓	✓	*			
FDA-approved and preferred	FDA-approved & acceptable	FDA-approved, but not recommended due to inferior performance		Lab-Developed Test (LDT), not FDA-approved		

***LAB ONLY:** Within 24 hours of collection, aliquot the specimen from the sterile container into a **yellow Aptima® Urine Tube**. Do not overfill or underfill.

****LAB ONLY:** Prior to cytology processing, aliquot the specimen from ThinPrep® container into a **green Aptima® Specimen Transfer Tube**.

Additional collection guidelines to avoid specimen rejection:	Match the sample type to the correct collection kit .	Pink swab = orange tube Blue swab = white tube	Discard/do not send the white cleaning swabs to the laboratory.	A maximum of two tests per collection kit are allowed. <ul style="list-style-type: none"> • (1) Beaker label per kit • (1) swab per kit